# Fiche technique

## Technical notice

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**Description:**  
Containment Mask, non-sterile

**Composition:**  
100% Polypropylene nonwoven - 3 Ply

**Level of filtration and desirability:**  
The test carried out on 04/15/2020 by the DGA demonstrated good performance in terms of particles filtration (aerosols) and air permeability (desirability). Tests carried out by the IFTH (French Textile and Clothing Institute) have shown that this mask can be washed and reused 5 times. This mask respects the French standard AFNOR S76-001.

**Recommendations for use:**  
Before using the mask, wash your hands with soap and water or with a hydroalcoholic solution. Apply the mask to cover the nose and mouth and adjust the rubber bands behind the ears. Bend the metal clip to fix the mask to the nose. When it gets wet, replace it with a new mask. To remove the mask remove it from behind (do not touch the front of the mask), throw it in a closed bin and wash your hands with soap and water or using a hydroalcoholic solution.

**Washing the mask:**  
Before washing the mask, it is recommended to clean your washing machine, by performing a cold rinse with bleach or launch an empty machine at 60 ° C or 95 ° C without spinning. Use usual detergent, making sure beforehand that it is not toxic by inhaled residues. The complete washing cycle (wetting, washing, rinsing) must be at least 30 minutes with a washing temperature of 60 ° C. The use of fabric softener is not recommended.

**Drying of the mask:**  
For domestic drying, use a dryer and clean the filters (and wash your hands afterwards). In laundry, dryers should be used. Barrier masks must be dried completely (all layers). Microwave cleaning is not recommended. A visual inspection must be carried out after each washing cycle. If any damage to the mask is detected (less adjustment, usury, deformation, etc.), the mask should be discarded.

**REMINDER:** This mask can be used 5 times maximum if washed and dried between each use. After those 5 uses, the mask must be thrown in a closed bin.

Caractéristiques Techniques   Technical Characteristics	Grammage   Weight
Face Externe   Outer Layer	40gr/m <sup>2</sup> ± 2gr/m <sup>2</sup>
Filtre - Spunbond   Filter - Spunbond	20gr/m <sup>2</sup> ± 2gr/m <sup>2</sup>
Face Interne   Inner Layer	40gr/m <sup>2</sup> ± 2gr/m <sup>2</sup>

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